Automatic Upper Arm Blood Pressure Monitor

Quick Guide

Model: BP2/BP2A


Contact us: info@med100.de

Website: www.med100.de

English

Thank you for purchasing the BP2/BP2A blood pressure monitor. For more details about the product, please refer the user manual.

1. Overview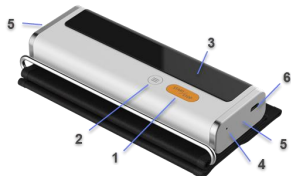


1. Start/Stop button
2. Function button
3. Display screen
4. LED indicator
5. ECG electrode
6. Charging cable connector

2. Blood Pressure Measurement

2.1. Applying the Arm Cuff


Remove tight-fitting clothing from your left upper arm. Do not place the arm cuff over thick clothes. Wrap the cuff around the upper arm, about 1 to 2 cm above the inside of the elbow, as shown.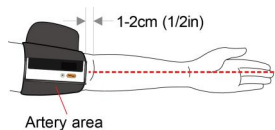

Note: Keep the monitor wearing position aligned with the middle finger.

2.2. How to Sit Correctly

Place your arm on a table so the cuff is level with your heart.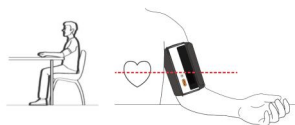

2.3 Measuring Blood Pressure

1. Power on the blood pressure monitor.
2. Press **Start/Stop** button to start blood pressure measurement.
3. The monitor will automatically deflate the cuff slowly during measurement.
4. The readings will be displayed when the measurement finished.

Note: During measurement, you should keep still

and do not squeeze the cuff.


Blood pressure result screen

3. Recording ECG


There are four methods for recording ECG by cablefree operation.

Note: The recording ECG feature is only available for the BP2 model.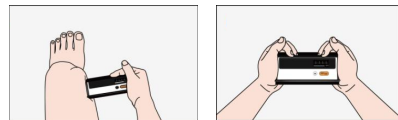

A. Right hand to left leg

B. Hand to hand

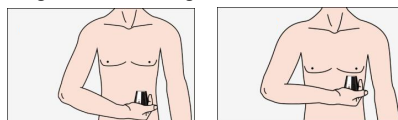C. Right hand to abdomen D. Right hand to chest

To start an ECG Recording:

1. Put your right palm on the right side electrodes of the Monitor.
2. Put the left side electrodes to the body position that you desired to measure.
3. Once the body parts are placed on the electrodes, press Function button to start the EKG recording.
4. The readings will come after 30 seconds.

Recording ECG screen

Note: You can get different signal amplitude from the different methods. Use method A or method C if signal is too low in method B.

4. Download the App

App name: ViHealth

iOS: App Store

Android: Google Play

Caution: DO NOT pair the device in your smart device's settings.

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